Omnifill C Caps

We care. You smile.

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EN Instructions for use

[MD] EU Medical device
In accordance with DIN EN ISO 9917-1

Product description:

Omnifill C Caps is a radiopaque glass ionomer cement for fillings in application capsules for direct application. Omnifill C Caps adheres chemically to enamel and dentine and continuously releases fluoride ions.
From every application capsule 0.15 ml cement can be extracted.

Indications:

- Restoration of V-shaped defects and cervical enamel or root erosions including class V cavities
- Restoration of deciduous teeth (especially class I)
- Fillings of class II cavities (not occlusion-bearing) by means of tunnel preparation
- Build-up fillings and linings, core build-up
- Temporary long-term treatment of class I and II cavities

Contraindications:

Omnifill C Caps contains polyacrylic acid. Omnifill C Caps should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:

Omnifill C Caps is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

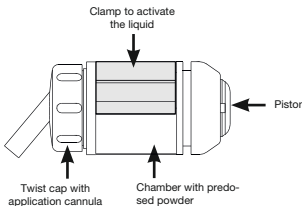
Omnifill C Caps should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Prepare the cavity using a minimally invasive approach (removing only carious tooth substance). The layer thickness of the restorations must not be less than 1 mm. Avoid thinly tapering margins. Clean the cavity, rinse with water and dab or blow dry. Do not overdry the dentine: it should have a moist, shiny appearance. If the residual dentine thickness is insufficient, the area close to the pulp should be covered with a suitable calcium hydroxyde product.

The capsule must be activated, mixed and opened before application.

Activation:

For activation, the liquid from the sachet underneath the clamp has to be emptied as completely as possible into the inner chamber of the capsule. This is done with a capsule activator. Place the capsule in the activator so that the application cannula protrudes from it and the clamp faces the moveable part of the activator. Hold the lever for 2 s in completely compressed position.

Mixing:

The capsule must only be activated directly before use and loaded immediately into a high-frequency mixer with at least 4,000 oscillations per minute. Mix the activated application capsule for 10 s, remove it from the mixer without delay and insert it in the holder of an application device.
The use of capsule mixers of other manufacturers may result in shorter or longer mixing times due to different mixing frequencies.

Opening:

Open the twist cap with application cannula with a half turn to the right until you notice a stop (see stamp on protective cap).

Application:

Insert the opened capsule into the holding bracket of an applicator. By turning the capsule in the holding bracket the cannula can be aligned in the desired direction.
Compress the lever slowly and evenly to apply the material directly into the cavity within 30 s after completion of mixing. Do not apply excessive pressure.

Mixing time	Working time	Setting time
10 s	2 min	4 min

After mixing, Omnifill C Caps has a working time of approximately 1.5 min (room temperature). During the placing of a filling the working area must be kept dry. The use of a rubber dam is recommended. Glass ionomer cements are susceptible to humidity (breath, saliva) or, respectively, dehydration during the setting time, especially directly after application. If possible, cover immediately with a strip or matrix or apply a protective varnish.

Finishing and polishing (flexible polishing discs) of fillings should be conducted no earlier than 7 min after placement of the filling. Recommendation: Re-apply protective varnish afterwards.

Warnings, precautionary measures:

- Omnifill C Caps should be mixed only immediately before application in order to have the full working time available for inserting and shaping and to avoid cracks from working the material too late.
- Ensure that the application capsule is mixed immediately after activation.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Fluorosilicate glass, polyacrylic acid, water, fumed silica, tartaric acid, colour pigments

Storage instructions and application method:

Store Omnifill C Caps in a dry place at temperatures of 15 °C to 23 °C. Only remove it from the foil immediately before use. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Omnifill C Caps must be reported to the manufacturer and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of Omnifill C Caps can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>).

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MP Medical Product GmbH