## Omnitemp C

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### GB Instructions for use

Omnitemp C, a eugenol-free cement with calcium hydroxide, is easy to process and does not influence the setting of resin-based filling and luting materials. Omnitemp C contains calcium hydroxide to support tooth vitality.

#### Indications:

Omnitemp C is used as a temporary cement for the attachment and for sealing of small one-surface cavities.

#### Method of use:

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Place and lock the mixing tip with a 90° turn clockwise. The material is automatically mixed in the mixing tip. All syringes come with a detachable piston. The piston can simply be pulled from the syringe and re-placed without causing any material leakage. The plugs in the syringe chambers remain in place preventing cross-contamination of the materials.

#### Application:

Apply a thin layer of Omnitemp C inside the temporary crown or bridge and affix it to the prepared tooth. Omnitemp C remains workable for about 1 min. The material sets in the mouth within 4 - 6 min. Place the temporary into the mouth. Remove any excess material after 3 - 5 min.

#### Information, precautionary measures:

In case of opened pulp or inferior dentine thickness, an adequate pulp protection is recommended. Omnitemp C contains mod. resin. Omnitemp C should not be used in case of known hypersensitivities (allergies) to this ingredient. The sensitivity of cores can be reduced by use of desensitizing preparations prior to temporary luting. Remove these preparations completely before definite luting (do not use preparations containing eugenol!). Extensive superficial and multisurfaced cavities as well as temporary longterm sealing work should be treated with a harder cement. The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by OMNIDENT. All the mixing tips for use with our respective products are available from OMNIDENT.

#### Supplementary indications:

Residues of Omnitemp C can be removed from crowns easily after they have been sprayed for a short time with a cooling spray. Store Omnitemp C at temperatures between 4° C - 23° C. Omnitemp C corresponds to DIN EN ISO 3107. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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MP Medical Product GmbH