OmniBiozid

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GB Instructions for use

Type of application:

As per 93/42/EEC: Water-based ready-to-use surface disinfection for the prophylactic wipe disinfection of non-invasive medical devices. Particularly suitable for use on alcohol-sensitive surfaces such as acrylic glass.

As per 528/2012/EC: Water-based ready-to-use wipe disinfection for the disinfection of surfaces. Particularly suitable for use on alcohol-sensitive surfaces such as acrylic glass.

Application:

Apply ready-to-use solution undiluted from a distance of approx. 20-30 cm as foam (use a suitable foam spray head) onto the surface to be disinfected. Distribute evenly with a lint-free, uncoloured disposable wipe and let it take effect. Ensure complete coverage of the surface. If necessary, remove excess product with an uncoloured disposable wipe.

Before applying the product on large-areas of sensitive materials such as artificial leather, first test for material compatibility on a small hidden area. To prevent streaks, the treated areas should be wiped with a damp (water) disposable cloth at regular intervals. The disinfectant solution may only be used for impregnating wipes compatible with the solution as per EN 16615.

Application times of the impregnation liquid:

Bacteria and Candida albicans as per EN 13727, EN 13624, EN 1276, EN 1650 (dirty condition): Conc. 1min., Bacteria and Candida albicans as per EN 13697 (dirty condition): Conc. 2min., Enveloped viruses (e.g. HBV, HIV, HCV) as per RKI (Robert Koch Institute) / DVV (German Association for the Control of Virus Diseases), dirty conditions: Conc. 1min., Rotaviruses as per EN 14476 (clean conditions): Conc. 1min., Noroviruses as per EN 14476 (clean condition): Conc. 30min., Polyomaviruses (SV40) (RIKI/DVV with contamination): Conc. 30min.

Application times for wipe systems:

Bacteria and Candida albicans as per VAH (Association for Applied Hygiene) with mechanic and EN 16615 (dirty conditions): Conc. 2min.
Bacteria and Candida albicans: VAH-list entry: Conc. 5min. (shortest application time for surface disinfection in VAH-list)

100g contains:

0.24g Benzyl-C12-16-alkyldimethylammonium chloride

0.24g N,N-didecyl-N,N-dimethyl ammonium chloride

0.24g C12-14-alkyl[(ethylphenyl)methyl]dimethyl ammonium chloride

Only put the packaging into the recycling system if it is empty of residue. Dispose of residual product in accordance with EAK 07 06 99.

648/2004/EC: Contains < 5% cationic surfactants, fragrances.

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