Omnisuc

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GB Instructions for use

Type of application:

For cleaning and disinfecting dental suction systems (incl. suction hoses), amalgam separator

Notes on application:

The concentrate must be first diluted with water. Recommended application concentration: 2%. Add tap water to 20 ml of concentrate to a total of 1.000 ml and aspirate the solution through the suction hoses (saliva ejector). Depending on the condition of the suction system, disinfect it once or twice a day. Disinfect the spittoon by pouring in approx. 250ml of the ready-to-use dilution. Do not mix with detergents or other disinfectant! Always prepare ready-to-use solutions fresh and use immediately! For professional users only.

Application times:

Bacteria and Candida albicans as per VAH in dirty conditions as an instrument disinfectant: 2.0 % 5 min.; 1.0 % 60 min.
Bactericidal (incl. MRSA) acc. EN 13724, EN 14561, VAH (dirty conditions): 3.0 % 2 min.
Yeasticidal (Candida albicans) acc. EN 13624, EN 14562, VAH; partially virucidal (Enveloped viruses e.g. HBV, HIV, HCV) acc. DVV/RKI (dirty conditions): 3.0 % 2 min.
Enveloped viruses (e.g. HBV, HIV, HCV) as per RKI/DVV (with contamination): 2.0 % 2 min.

Safety Instructions:

H 314 Causes severe skin burns and eye damage. H 411 Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Dispose of contents/container in accordance with all local, regional, national and international regulations. Contains N,N-dimethyl-N-dioctylammonium chloride, potassium hydroxide. (EC) 648/2004: < 5% nonionic surfactants, < 5% cationic surfactants, fragrances.

Notes on incidents:

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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