Omnisuc

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GB Instructions for use

Areas of application: High performance solution designed for cleaning and disinfecting of dental Evacuation systems (incl. suction hoses and amalgam separator) and spittoons.

Indications for use: The concentrate must be first diluted with water. Recommended application concentration: 2%. Add 20 ml concentrate with tap water to 1.000 ml and aspirate the solution through the suction hoses. Regularly, the suction system should be disinfected once or twice a day. The spittoon is disinfected by purging approx. 250 ml of the ready-to-use dilution. Do not mix with other disinfectants or cleansers! Always prepare ready-to-use solutions fresh and use immediately.

Range of activity: bactericidal (incl. MRSA) acc. EN 13724, EN 14561, VAH; yeasticidal (Candida albicans) acc. EN 13624, EN 14562, VAH; partially virucidal (Enveloped viruses e.g. HBV, HIV, HCV) acc. DVV/RKI.

Effectivity: Bacteria and Candida albicans in accordance to VAH (high contamination): 2,0% 5min.; 1,0% 60min.

Bacteria in accordance to EN 13727 and EN 14561 (high contamination): 3,0% 2min.
Candida albicans in accordance to EN 13624 and EN 14562 (high contamination): 3,0% 2 min.
Enveloped viruses (i.e. HBV, HIV, HCV) in accordance to BGA (RKI)/DVV (contamination): 2,0% 2min.

Safety Instruction: H 314 Causes severe skin burns and eye damage. H 411 Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection.
IF ON SKIN: Wash with plenty of water.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

If skin irritation occurs: Get medical advice/attention.

If eye irritation persists: Get medical advice/attention. Dispose of contents/container in accordance with all local, regional, national and international regulations. Contains N,N-Dimethyl-N-Dioctylammoniumchlorid, Kaliumhydroxid. VO 648/2004/EG: < 5% nonionic surfactants, < 5% cationic surfactants, fragrances.

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PRISMAN GmbH