

Helix Test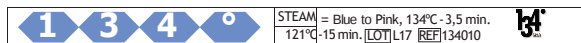

Referenz / Reference / Référence / Referenze / Referencia

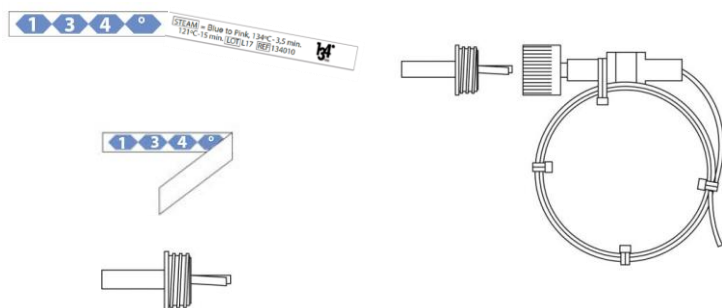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GB MANUAL

1. Make sure the Helix is dry and does not show any visual damage.
2. Unscrew the cap and remove the indicator holder by unscrewing it.
3. Tear off an indicator strip from the sheet. Fold the indicator strip at the center, with the indicator on the inside.
4. Insert the strip into the indicator holder, folded side first. **Important:** If this process is not carried out in accordance with these instructions, the indicator's colour change and results may be affected.
5. Make sure that the indicator does not slide out of the indicator holder.
6. Now reassemble the test specimen as shown in the figure.
7. Run a sterilisation cycle as described on the indicator.
8. After sterilisation, remove the Helix from the steriliser.
9. Open the indicator holder and remove the indicator. The batch was successful if the indicator colour changes according to the reference.
10. The indicator strip has been provided with a self-adhesive backing so it can be attached to a batch record. This procedure prevents human error and allows for better traceability. Indicators must be stored in a cool, dry place both before and after use!

! The Helix test checks the sterilization conditions at the location where it is placed in the sterilizer. Ask the manufacturer of your sterilizer for the place where the most difficult conditions are. Often this is close to the door.

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