- D

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GB

Manual

F

Mode d'emploi

E

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I

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NL

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S

Bruksanvisning

DK

Brugsanvisning

PL

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RUS

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Omnicent®

Silicone Moulding Compounds, Normal and Hard, Liquid Hardener and Paste Hardener

Rev. 01.2007

Silicone-based mouldable precision impression materials (polysiloxane, condensation-curing)

Conforms to EN ISO 4823 Type O, very high consistency, mouldable

Examples of applications for OMNIDENT Normal and Hard:

- First impression for double impression technique
 - Impressions in situ
 - Occlusion
 - Tray material for the sandwich technique

OMNIDENT Hard can also be used as outer rim material or for making small models

Amount required	Liquid hardener	Paste Hardener
1 spoonful of normal moulding compound <i>(12 ml / 19,3 g)</i>	10 drops <i>(0,19 ml / 0,2 g)</i>	2,5 cm <i>(0,5 g)</i>
1 spoonful of hard moulding compound <i>(12 ml / 19,6 g)</i>	10 drops <i>(0,19 ml / 0,2 g)</i>	2,5 cm <i>(0,5 g)</i>

Storage of the impression: Store in a dry place at room temperature

Model fabrication:

Impressions should be cast 15 minutes after removal from the mouth at the earliest. The best results are achieved within the first week.

Electroplating: Impressions can be electroplated with copper or silver in conventional baths.

Disinfection: Impressions can be disinfected with Omnisep IMP, for example (DGZMK 3/93 V1.0).

Work and safety instructions for moulding compounds, hardener paste and hardener liquid:

Flammable, keep away from ignition sources.

Inhaling and swallowing can cause damage to health. After swallowing: consult a doctor immediately.

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Avoid eye contact with hardener, protective goggles are recommended. After eye contact: rinse the eyes under running water for several minutes keeping the eyelids open.

If the complaints persist, consult a doctor.

Avoid skin contact, wear gloves. After skin contact: wash with soap and water immediately and rinse thoroughly. If skin irritation persists, consult a doctor.

Storage:

Do not store above 25°C.

Shelf life:

See date on pack.

Technical data:

Omnicent Moulding Compound	Colour	Mixing time	Minimum Total Working Time* (liquid hardener)	Minimum Total Working Time* (paste hardener)	Minimum Time in the Mouth	Deformation under pressure	Elastic recovery after deformation	Linear dimensional change	Hardness
		Min : sec	Min : sec	Min : sec	Min : sec	%	%	%	Shore A
Normal	White	00:30	01:45	02:00	04:00	< 3.5	≥ 98	≤ 0.5	65 ± 5
Hard	Yellow	00:30	01:45	02:00	04:00	< 3.5	≥ 98	≤ 0.5	70 ± 5

* Ambient conditions: 23°C, 50 ± 5% rel. humidity.

Higher temperatures speed up setting whilst lower temperatures slow it down.

The processing, use and application of products we supply is solely your responsibility because we do not have any control over conditions. In case where we provide applicational advice orally, in writing or by conducting tests, the advice is to be regarded as non-committal information, also in respect of any third-party property rights. It does not relieve you of your duty to conduct your own tests on the products supplied with regard to their fitness and suitability for the intended purposes and procedures. With regard to all circumstances and all claims our liability shall be limited to the value of the goods which we have delivered and you have used. We give a warranty for the defect-free quality of our products in accordance with our standard terms and conditions.

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