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Tooth Whitening System for use in customized trays. Store in a cool, dark place. Do not freeze. Contains hydrogen peroxide. Keep out of the reach of children.

Avoid contact with eyes. Rinse immediately if product comes into contact with them. Not to be used on a person under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use. With Omni White smile you can whiten your teeth in a safe and effective way without causing damage.

Please be aware that crowns, bridges and fillings will not benefit from any bleaching procedure. A professional tooth cleaning is recommended before using this product. The teeth should be in good condition and without cavities. Please consult your dentist if you have questions.

Omni White smile is available in various concentrations. The higher the carbamide peroxide content is, the shorter the treatment time. The treatment can be done overnight or for 1–4 hours during the day. The treatment should be repeated over 5–10 days.

Carbamide-Peroxide concentration	Minimum time of application	Recommended time of use	Best result for regular discolouration	Indications
10% CP (~ 3,5% HP)	2 hours	4 hours or overnight	5–10 applications	Minor to regular discolouration, sensitive teeth
16% CP (~ 5,6% HP)	1 hour	3 hours or overnight	4–8 applications	Minor to regular discolouration, sensitive teeth

Recommended Treatment times

The indicated times are only recommendations. A longer wear time per session is possible and can reduce the number of applications. In case of sensitivity a shorter wear-time of the trays with more applications is recommended and possibly a lower concentration of carbamide peroxide gel.

How to achieve maximum results

- Brush and floss your teeth.
- Take one syringe out of the box and remove the cap from the dispensing tip.
- Place a small drop of Omni White smile Gel into each tooth compartment (Pic.1). Place the filled trays on your teeth (Pic.2). One syringe fills approximately 6–8 trays.
- After whitening, remove the trays, rinse and brush excess gel from your teeth and gums. Rinse the trays with cold water and clean gently with a toothbrush.
- For at least 4 hours after bleaching, please refrain from consuming tea, coffee, red wine, cigarettes etc. as this will reduce the effectiveness of the treatment.

Possible side effects

Tooth sensitivity, gum irritation, nausea, headache. Symptoms are temporary and should subside in 1–3 days following termination of product use.

Warning

Avoid contact with the inside of the mouth, eyes and skin. If accidental contact occurs, rinse affected area generously with water. In case of contact with eyes, rinse immediately with water and seek medical advice. Omnident does not recommend bleaching treatments on people who are pregnant, lactating.

Ingredients:

Glycerin, Carbamide Peroxide, Carbomer, Disodium Pyrophosphate, Sodium Stannate, Dipotassium EDTA, PEG-40 Hydrogenated Castor Oil, Tromethamine, Aroma.

WHITHEsmile GmbH
Weinheimer Straße 6
D-69488 Birkenau

OMNIDENT Dental-
Handelsgesellschaft mbH
Gutenbergring 5
D-63110 Rodgau