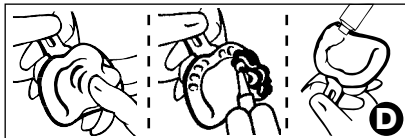
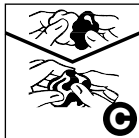
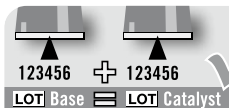
|           |                            |    |
|-----------|----------------------------|----|
| <b>DE</b> | Gebrauchsanweisung .....   | 5  |
| <b>GB</b> | Instructions for use ..... | 9  |
| <b>FR</b> | Mode d'emploi .....        | 13 |
| <b>IT</b> | Istruzioni per l'uso ..... | 17 |
| <b>NL</b> | Gebruiksaanwijzing .....   | 21 |
| <b>PL</b> | Instrukcja obsługi .....   | 25 |

|    |    |
|----|----|
| EC | MD |
|----|----|

Medizinprodukt  
Medical device  
Dispositif médical  
Dispositivo medico  
Medisch hulpmiddel  
Wyrób medyczny

**A**

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**C** = Mischzeit

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**H** = Intraoral Tip

Elastomeric, silicone-based precision impression material, addition-cured

**All products have a dosing ratio 1 : 1 (volume or weight).**

#### **Practical tips**

We do not recommend use in combination with any other silicone impression material. Visually inspect for damage before use. Damaged products must not be used. For use by dental personnel only.

**Tubes and Tubs must be carefully sealed after use. Do not inter-change lids, caps or spoons. When using cartridges extrude some material before positioning mixing tip. After use leave used mixing tip in situ as sealer on cartridge. Before next use remove sealed tip. Extrude some material and then fix new mixing tip. Mixing Tips and Intraoral Tips are single-use products. To avoid cross contamination Mixing Tips and Intraoral Tips must be disposed of immediately after initial use.**

With addition-cured silicone impression materials changes in setting characteristics can occur if the material comes into contact with latex gloves, retraction solutions or freshly placed composite fillings.

|                      | EN ISO<br>4832 type                        | Color Base/<br>Catalyst | Mixing time<br><br>min : sec | Min.<br>Working<br>time*<br><br>min : sec | Min.<br>Time in<br>mouth<br><br>min : sec | Mixing Tips | Intraoral<br>Tips |
|----------------------|--------------------------------------------|-------------------------|------------------------------|-------------------------------------------|-------------------------------------------|-------------|-------------------|
| Putty                | Type 0<br>putty<br>consistency             | yellow/grey             | 00:30                        | 02:00                                     | 03:30                                     | yellow      | yellow            |
| Dynamik<br>Putty 380 | Type 0<br>putty<br>consistency             | yellow/grey             | -                            | 01:30                                     | 02:30                                     | yellow      | yellow            |
| Kontur               | Type 2<br>medium-<br>bodied<br>consistency | light blue/<br>grey     | -                            | 02:00                                     | 03:30                                     | yellow      | yellow            |

\*at room temperature of 23°C (73°F), 50% rel. humidity. Higher temperatures shorten, lower temperatures prolong these times.

|                                    |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
|------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Contraindications:</b>          | The use of this product is contraindicated in case of known or suspected allergies against components of this product.                                                                                                                                                                                                                                                                                                                                                                                                                                               |
| <b>Side effects:</b>               | This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer. Omnisil A contains surfactants for hydrophilation, which in some patients may cause a superficial anesthetic effect. Surplus material on the impression tray or in the patient's mouth should be avoided as this may result in gagging, swallowing or aspiration. To further reduce these risks during impression taking, the patient should be positioned as upright as possible. |
| <b>Model fabrication:</b>          | The impression can be cast 60 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.                                                                                                                                                                                                                                                                                                                                                                                                             |
| <b>Recommended model material:</b> | All standard model materials can be used.<br>We recommend: Class III Gypsum, e. g. Omnidanos;<br>Class IV Gypsum, e. g. Omni Stone Superhartgips                                                                                                                                                                                                                                                                                                                                                                                                                     |
| <b>Disinfection:</b>               | Impressions can be treated with Omnisept IMP or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.                                                                                                                                                                                                                                                                                                                                               |

**Adhesion:**

We recommend using our Omni Löffeladhäsiv Silikon.

**Warning and precaution:**

Avoid contact with skin and eyes. May be irritating to eyes and skin. Polyvinyl siloxanes are chemically resistant. Material which has not set can soil clothing. **Wear protective gloves according to EN 420, category III, with protection against chemical risks according to EN 374, type C.** On contact with skin wash off with water and soap. If in eyes rinse with water for several minutes. Contact a physician in case of persistent complaints. Do not swallow or ingest. Do not leave any residual material in the sulcus or oral cavity, as this can lead to abscess formation. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases.

**Storage conditions:**

Do not store above 25°C (77°F). Do not use the material after the expiration date. Please quote batch number and the article number in all correspondence about the product. According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred. For disposal please follow the instructions on the material safety data sheet and national regulations.

Dated: 2019-09

**Description of applicational / packaging icons**

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